


Preg

PREGNANCY TEST

Preg is an accurate and easy to use home pregnancy test. Please read these instructions carefully before use. Do not open the foil pouch until you are ready to perform the test. Have a watch, clock or timer available.

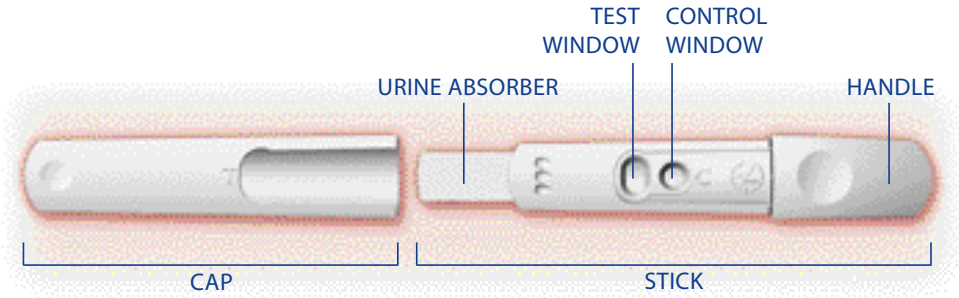


HOW TO PERFORM THE TEST

Preg can be used at any time of the day, from the day your period is due. Take the pregnancy test out of the foil pouch. Open the test by removing the cap from the stick.



Hold the urine absorber downward in your urine stream for at least 5 seconds until it is wet. Continue to hold the stick pointing downwards.

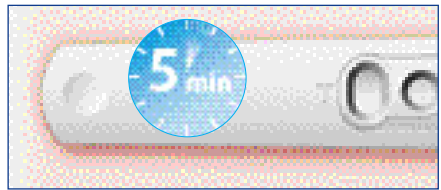

Replace the cap back on the stick. Lay the stick down flat with the windows facing you and wait 5 minutes before reading the result.

HOW TO READ THE RESULT

After you have completed the test, whether you are pregnant or not, a pink/purple line will appear in the round control window. This indicates that the test is working properly.

Read the result 5 minutes after you have conducted the test. Do not interpret the result after 30 minutes.



YES
Pregnant

You are pregnant if a pink/purple line appears in the large oval test window (T). Even if the line is very pale, it still means you are pregnant. A positive result may in some cases be observed in less than 1 minute.



NO
Not pregnant

You are not pregnant if no pink/purple line appears in the large oval test window (T).



Important: There should always be a control line visible. If there is no control line, the test has not worked properly and the test result is invalid. There might have been insufficient urine on the absorber. Repeat the test with a new device, following the instructions carefully.

POSITIVE RESULT? KEEP THE TEST!

A positive test result will remain unchanged and some people like to keep a positive test to show to loved ones or even put in baby's first album. Preg has optimised its design: one click and the handle is removed, which makes the test more compact and easier to store.



FREQUENTLY ASKED QUESTIONS ABOUT PREG PREGNANCY TEST

How accurate is Preg?

Preg has been proven to be over 99% accurate, and as reliable as any pregnancy test performed by a doctor.

How does Preg work?

When a woman becomes pregnant, her body starts to produce a special hormone known as hCG (human Chorionic Gonadotrophin), which is present in the urine and increases daily. Preg is uniquely designed to detect very small amounts of this pregnancy hormone in your urine, so you can test from the day that your period is due.

How soon can I use Preg?

Preg can detect hCG in your urine with over 99% accuracy as early as the day your period is due. If you have irregular periods, to work out when to test, take your longest cycle and add one day, to avoid testing too early.

Testing earlier than the day your period is due is possible, but not recommended, as the result will not be over 99% reliable. If you test earlier, you might be pregnant but still not have enough hCG produced to be detected by the test. It is advisable to wait until the first day your period is due before testing.

Do I have to test in the morning?

No, you can use Preg any time of the day. Excess fluid intake immediately before testing may dilute the pregnancy hormone so drink normally before you test.

After 5 minutes, the test window had no line, but some minutes later it appeared.

If the line appears in the test window within 30 minutes, the result is positive; no matter how pale the line is.

How long will my result remain before it fades away or changes?

A positive result will remain unchanged, however the background may become darker. Positive results can be kept. With a negative result, a very faint line may appear in the

test window after more than 30 minutes. This does not indicate a positive result. Remember do not interpret the result after 30 minutes.

Can I still use the test if I collect a urine sample in a cup?

Yes, as long as the container used is clean and dry. Do not use detergent or solvent to clean it. Test right away: immerse the urine absorber in the urine for at least 10 seconds. Pay attention that the urine level does not go above the plastic casing of the stick. Wait for 5 minutes before you read the result.

What do I do if I get a positive result?

If the test is positive, we recommend you consult a doctor to discuss your pregnancy and what steps and precautions you should take next. Early prenatal care is very important to ensure the health of both you and your baby.

The test result says I am not pregnant, but I am still not sure?

If the result says you are not pregnant, no pregnancy hormone (human Chorionic Gonadotropin) has been detected. You may have miscalculated the day your period was due. If your period has not started within a week, repeat the test using another Preg home pregnancy test. If the result is negative and you have still not had your period, consult your doctor.

Can any medication affect the result?

Preg is only affected by fertility drugs which contain hCG (human Chorionic Gonadotropin) or hMG (human Menopausal Gonadotropin). Other fertility drugs (such as Clomid), antibiotics, painkillers or the contraceptive pill, should not affect the result.

Can I use the Preg test more than once?

No, you can use each test only once.

ANY FURTHER QUESTIONS?

ACO HUD NORDIC AB

Email: info@aconordic.com

Telephone: +46(8) 59 00 29 00

In correspondence, please quote the lot number printed on the pack.

www.aconordic.com

Composition: test stick containing coated membrane with monoclonal and polyclonal antibodies and a pad containing monoclonal IgG (antibody) dye conjugate in protein matrix.

Store at room temperature (15°C – 28°C) in a sealed pouch. If foil is damaged, do not use the test.

Keep out of reach of children.

2 Use only once.

Read instructions carefully before use.

LOT Lot number.

Use before expiration date, printed on the box.

IVD In-Vitro Diagnostic Medical Device.

Manufacturer: Omega Teknika - Farnham Drive - Finglas - Dublin 11 - Ireland

Distributor: ACO HUD NORDIC AB, SE - UPPLANDS VÄSBY, NO - OSLO, DK - HORSHOLM, FI - ESBOO

Date of revision 08/2007

26112

